

AUSSIE LIFTS

THE HANDY-LIFT



Providing an alternative to negotiating the stairs in Recreational Vehicles such as:

- **Caravans**
- **RV's**
- **Domestic Buses**
- **Mobile Homes**



- Quiet in Operation
- Remote Control with Push Buttons and Extendable Cord
- Electric Powered by 12 Volt DC
- Approx 60 seconds to lift 136kg to a height of 1000mm
- Folds up and locks away when transporting and not in use
- Designed for Mobility Use - This model is not suitable for wheelchair use

12 Month Warranty


Ph: 07 3274 4211 Fax: 07 3274 5543
12 Achievement Crescent, Acacia Ridge, QLD 4110
www.aussielifts.com.au

AUSSIE LIFTS

THE HANDY-LIFT

Easy To Use

The simplistic design of the Handy-Lift transfers through to its operation which offers usage by a single user or attendant via the push button remote control.

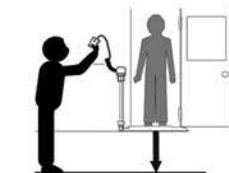



1. Lower platform, with passenger in center of platform, all the way to the ground.



2. Close outside door. Raise empty platform all the way up.



3. Remove latch pin and rotate platform toward the actuator. When completely rotated, replace latch pin.

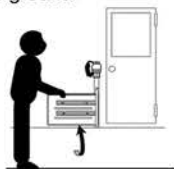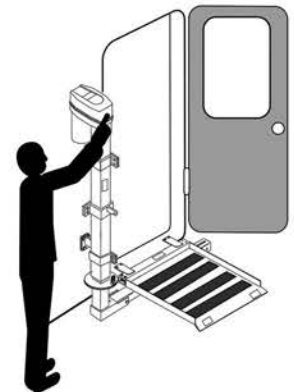

4. Fold platform in upright position.



5. Raise platform till it latches securely.



1. Lower platform to unlatch it.




2. Unfold platform.




3. Pull latch pin and rotate platform. Lower platform to the ground. Be sure latch pin is open outside door. place after rotating platform.



4. Lower platform to the ground. Be sure latch pin is open outside door. place after rotating platform.

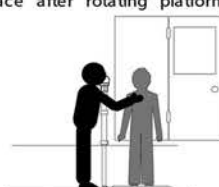

5. Passenger must always stay in the center of the platform.

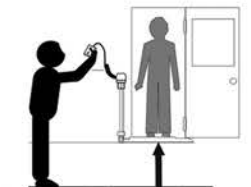

6. Raise platform, with passenger, all the way up.

Lifting Capacity: 136kg
Platform Width: 648mm
Platform Depth: 660mm
Travel: 1016mm

Handy-Lift

12 Month Warranty

Ph: 07 3274 4211 Fax: 07 3274 5543
12 Achievement Crescent, Acacia Ridge, QLD 4110
www.aussielifts.com.au